PORTUGUÊS 

Plenum Guide 3.0

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ENGLISH 

Plenum Guide 3.0

Instructions for Use

ESPAÑOL 

Plenum Guide 3.0

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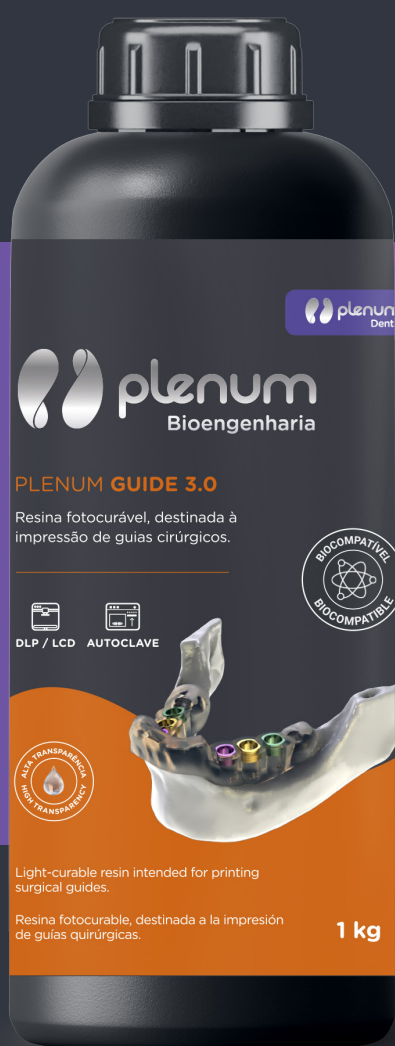
Rev00

Plenum Guide 3.0

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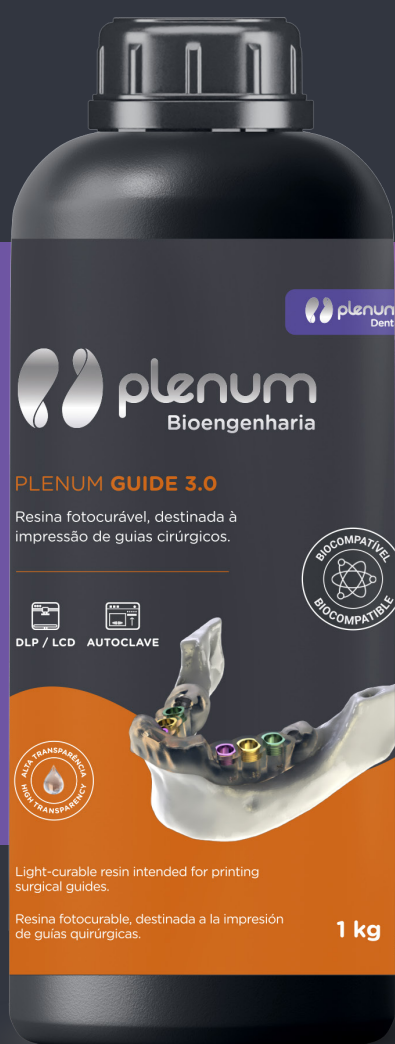
Plenum Guide 3.0

Instructions for Use

Power, endurance,
and control.
No compromises.

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Contraindications

This material is contraindicated for any purpose other than its intended use and when there is direct intraoral contact with resin that is not fully cured.

Device description

Plenum Guide 3.0 is designed for printing surgical guides in general and should be used exclusively for professional dental work.

Manufacturing process

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1. CAD file: File in STL format.

2. Prepare:

- 2.1** Make sure the printer you are using is compatible with Plenum Guide 3.0.
- 2.2** Set the recommended settings for the printer.
- 2.3** Sprinray Printers (Pro 95, Pro 55, and Pro 2): Use the Sprinray Surgical Guide 3 print profile.

3. Shake:

- 3.1** Shake the bottle well before use (2-5 minutes).
- 3.2** Pour the resin into the tank until it reaches at least the minimum level mark.
- 3.3** Gently mix the resin in the tank before each print.

4. Printing:

- 4.1** Print the selected documents.

5. Remove:

- 5.1** Remove printed objects from the platform immediately after printing is complete.
- 5.2** Remove all supports using a cutter or cutting disc, if necessary.

6. Cleaning:

- 6.1** Isopropyl alcohol, 91% or higher.
- 6.2** Ultrasonic bath (Duration: 2 cycles, 3 minutes each).
- 6.3** Pro Wash S or Pro Wash/Dry: We recommend using cycles of 3 to 4 minutes at a time.

7. Drying:

- 7.1** Dry the printed devices with compressed air.
- 7.2** If compressed air is not available, let the devices dry naturally.

8. UV Post-Curing:

- 8.1** Complete polymerization in the post-curing unit is essential for achieving the desired material properties.
- 8.2** 60 minutes in a 405-nm UV post-curing unit.
- 8.3** Power of 90W or higher.
- 8.4** ProCure 1: Sprintray Surgical Guide 3.
- 8.5** ProCure 2: Sprintray Crown (Bego).
- 8.6** NanoCure: Sprintray Crown (Bego).

9. Sterilization

- 9.1** Wash the surgical guides with warm, soapy water, insert the rings into the holes in the guide, and autoclave at 135°C for 3–5 minutes.

Warnings and precautions

Plenum Guide 3.0 is non-toxic in its processed and cured form and is classified as a biocompatible material.

When rinsing with a solvent, do so in a well-ventilated area while wearing appropriate protective equipment.

- **Skin contact:** May cause skin irritation. If unprocessed resin comes into contact with the skin, wash thoroughly with soap and water. May cause an allergic skin reaction. If skin sensitization occurs, discontinue use. If dermatitis or other symptoms persist, seek medical attention.
- **Inhalation:** High concentrations of vapor may cause headaches, eye irritation, and/or respiratory irritation. If exposed to high concentrations of vapor or mist, move to fresh air. Use oxygen or artificial respiration as needed.
- **Eye contact:** Thoroughly wash the affected area with soap and water.
- **Ingestion:** Contact a doctor immediately.

Storage

- **Reusing Material:** The resin remaining in the resin tank can be reused. You can use a filter to ensure that the resin is free of cured particles to prevent print failures. The remaining material in the tank can be poured back into the resin bottle after filtering. This process can be repeated until the material in the bottle is completely used up. Please note that when reusing the resin, it must be filtered and poured back into the same bottle.
- Store Plenum Guide 3.0 at temperatures between 2°C and 28°C (35.6°F and 82.4°F) and keep it out of direct sunlight.
- Keep the bottle closed and/or the tank cap securely fastened when not in use.
- Allow the material to fully cure before disposal.
- Do not use Plenum Guide 3.0 after the expiration date printed on the bottle.

Do not use expired resin; print stability may be compromised if expired photoinitiators are not activated properly.

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Fabricante / Titular del registro: ASTRO SCIENCE DO BRASIL S.A.
Ventas y Distribución: M3 Health Industria e Comercio de Produtos Médicos, Odontológicos e Correlatos S.A